H315
H319
H334
H335
P261
P280
P304 + P340
P342 + P311
P501

Cleaner Kit (Chem & ISE) / Kit de Limpieza (Química E ISE) / Kit de nettoyage (Chim & ISE) / Temizleme kiti (Kim & ISE) / Kit di detergente (Chimico e ISE) / Kit de solução de limpeza (Química & ISE) / Cleaner (Chem & ISE) - комплект контейнеров / Zestaw czyszczący (Chem i ise) / 清洁剂（化学分析仪和 ISE）用品

Cleaner Kit – Chemistry & ISE

Each dual reagent wedge contains two cleaning solutions. The large section of the wedge contains 29 mL of 20% bleach; the smaller section of the wedge contains dry Pepsin that must be reconstituted before use. The diluent bottle contains 10 mL of 0.1N HCl and salts and is used to reconstitute the Pepsin.

Intended use

For use in cleaning the probe and the ISE sensor system of the Medica EasyRA Chemistry Analyzer. The cleaner in the large section of the wedge is used to clean the probe on a daily basis. The cleaner in the small section of the wedge is used to clean the ISE sensors on a daily basis.

Safety Precautions and Warnings

H315 - Causes skin irritation. H319 - Causes serious eye irritation. H334 - May cause allergy or asthma symptoms or breathing difficulties if inhaled. H335 - May cause respiratory irritation. P261 - Avoid breathing dust/fume/gas/mist/vapours/spray. P280 - Wear protective gloves/protective clothing/eye protection/face protection. P304 + P340 - IF INHALED: Remove victim to fresh air and keep at rest in a position comfortable for breathing. P342 + P311 - If experiencing respiratory symptoms: Call a POISON CENTER or doctor/physician. P501 - Dispose of contents and container in accordance with all local, regional, and national regulations.

Reagents

Large wedge section: 1% sodium hypochlorite Small wedge section: 292 mg of Pepsin (porcine) powder Diluent: 10 mL of 0.1N HCl and salts

Instructions for Handling, Storage and Stability

The cleaning wedge is stable to the expiration date on the label before the pepsin is reconstituted if stored at 4-25° C. The bleach section of the wedge (larger section) is ready to use. The pepsin powder in the small section of the wedge must be reconstituted with the Diluent before use.

Instructions for Use

Prior to use, the pepsin powder must be reconstituted.

Reconstitution: Add the complete contents of the 10mL HCL diluent bottle into the small section of the Cleaner wedge containing the pepsin powder. Mix well to reconstitute the pepsin. It will take about twenty minutes for the pepsin to fully dissolve. Inspect the small section of the wedge to be sure all the Pepsin powder has been dissolved. Store the Cleaning Wedge capped at 2-8° C when not in use. Once reconstituted and used on the Medica EasyRA Chemistry Analyzer, the cleaning wedge contents have a use life of 30 days.

It is recommended that the probe be cleaned both at the beginning of the day, and at the end of the day. The ISE sensor cleaning is recommended at the end of the day only. Remove the caps and place the cleaner wedge in the EasyRA reagent wheel located in the reagent area. Refer to the EasyRA Operators Manual for detailed instructions on performing the daily cleaning procedures. The cleaner wedge should be recapped when not in use and stored at 2-8° C. Note: Check the inside of the necks of the wedge for foam after removing the caps and placing the wedge on the analyzer. If there is foam, remove it with a clean swab or a disposable pipette before performing the test. Use separate clean swabs for R1 and R2.

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