

Surfactant Kit / Kit de surfactante / Kit du surfactant /
Sürfaktan Kiti / Kit tensioattivo / Kit Tensoativo /
Комплект сурфактанта / Surfaktantu / 表面活性剂 /

REF 10670

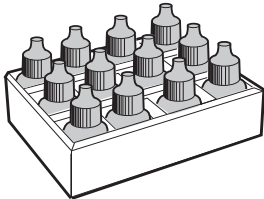12 x 10 mL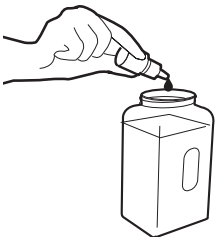

Surfactant

Dropper bottles each containing 10.0 mL of surfactant

Intended use

For use as an additive to the reagent-grade deionized water to be used as the Medica EasyRA Chemistry Analyzer diluent.

Safety Precautions and Warnings

Wear gloves and eye protection when handling this surfactant. Avoid eye or skin contact. Wash hands after using this material.

Instructions for Handling, Storage, and Stability

The surfactant is ready to use as supplied.

The Surfactant Kit is stable until the expiration date on the label if stored at room temperature, 18–25°C. The solutions should be clear and colorless. If turbid, this may indicate storage at a temperature below that listed on the labeling. Allow turbid surfactant to stand for one day at room temperature, 18–25°C and mix by inversion. Do not use if turbidity remains after one day at room temperature.

Do not refrigerate the surfactant.

Instructions for Use

Add 5 drops to each liter of EasyRA System Diluent reagent-grade water. The System Diluent bottle holds up to 4 liters of diluent (20 drops of surfactant per complete, filled diluent bottle).

The EasyRA diluent should be prepared on a daily basis in a volume that matches daily usage. Daily, discard any remaining diluent and rinse the container with deionized water before refilling with fresh water and adding the surfactant.

Refer to the EasyRA Operators Manual for complete instructions on filling the EasyRA diluent bottle.

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