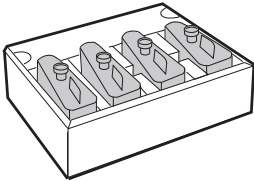
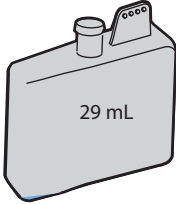
REF 10661

4 x 29 mL

Cleaner Kit (Chem) / Kit de limpieza (Chem) / Kit de nettoyage (Chem) / Temizleme kiti (Chem) / Kit detergente (Chem) / Kit de limpeza (Chem) / Очищающий набор (Хим) / Zestaw czyszczący (Chem) / 清洁剂试验盒 (化学分析仪)

Cleaner Kit—Chemistry

Wedges each contain a useable volume of 29 mL of 20% bleach.

Intended use

For use in cleaning the probe of the Medica EasyRA Chemistry Analyzer. The cleaner in the wedge is used to clean the probe on a daily basis.

Safety Precautions and Warnings

Use gloves when handling and dispose of in a safe manner. Avoid contact with eyes or skin. Wash hands after using this material.

Reagents

wedge: 1% sodium hypochlorite, 29 mL

Instructions for Handling, Storage and Stability

The bleach section of the wedge (larger section) is ready to use as supplied.

The unopened cleaner wedge is stable until the expiration date on the label if stored at 4°–25°C. Once it has been used on-board, store closed at 2°–8°C for up to 30 days.

Instructions for Use

The diluted bleach is ready-to-use. Store the Cleaner wedge closed when not in use, either on-board or in the refrigerator.

It is recommended that the probe be cleaned both at the beginning of the day, and at the end of the day. Remove the cap and place the cleaner wedge in the EasyRA reagent wheel located in the reagent area. Refer to the EasyRA Operators Manual for detailed instructions on performing the daily cleaning procedures. The cleaner wedge should be recapped when not in use.

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Medica Corporation, 5 Oak Park Drive