EasyQC®

Bi-Level Quality Control, Assayed  
Tri-Level Quality Control, Assayed

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TU

EN

Level 1 (Normal), Level 2 (Abnormal High),  
Level 3 (Abnormal Low)

Please refer to the Quality Control section in the  
Operator's Manual for detailed instructions. Observe  
all warnings and precautions stated in the manual  
involving the use of biological materials.

#### Intended Use

For in vitro diagnostic use only. To establish the  
performance of electrolyte analyzer systems.

#### Product Description

EasyQC controls are solutions of salts, buffers,  
bovine albumin and preservatives. EasyQC controls  
do not contain human serum or human serum  
products. No special precautions are required in  
handling the product other than those followed in  
routine handling of blood specimens.

#### Storage and Stability

Control material should be stored at 2° to 8°C and  
may be used to the expiration date on the bottle.

#### Use

- 1 Dispense control into sample cup, such as Medica  
REF 2843.
- 2 Analyze as a sample within 5-10 minutes to avoid  
evaporation.
- 3 Use of the Quality Control section of the Operator's  
Manual is recommended.
- 4 Recap bottle immediately after removing sample  
and refrigerate at 2° to 8°C.
- 5 IMPORTANT: Do not remove the dropper tip. Never  
insert any object into the control bottle.

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When the previous instructions are followed, these  
controls will be stable for 8 weeks from the date the  
bottle was first opened.

#### Limitation

These controls are for use as an aid in the  
evaluation of performance of electrolyte analyzers,  
including the EasyLyte and EasyElectrolytes  
Analyzers, and should not be considered as  
a substitute for other aspects of total quality  
control, such as calibration, maintenance, and  
record keeping. All levels of this control material  
have a clear light yellow color. A cloudy or turbid  
appearance may indicate product alteration.

#### Expected Values

Expected value ranges (values in mmol/L) for the  
specified lots of EasyQC controls are listed on the  
reverse side of this sheet. Approximately 95%  
of means from all EasyLyte and EasyElectrolytes  
Analyzers, operating according to specifications, are  
expected to fall within these ranges.

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