

MEDICA CREA-U CAL

REF 10652 1 X 10 mL BOTTLES

URINE CREATININE CALIBRATOR

Vials each containing 10 mL of Creatinine in 0.10N hydrochloric acid.

INTENDED USE

For use as the Urine Creatinine calibrator for the Medica Urine Creatinine test on the Medica EasyRA Chemistry Analyzer.

SAFETY PRECAUTIONS AND WARNINGS

Follow good laboratory practices..

INSTRUCTIONS FOR HANDLING, STORAGE AND STABILITY

The liquid calibrator is stable until the expiration date on the label if stored at 2-8° C.

After opening, the calibrator should be stored closed at 2 - 8° C.

INSTRUCTIONS FOR USE

Ready-to-use.

VALUE ASSIGNMENT

The Table below contains the assigned values for the Urine Creatinine Calibrator:

Lot No. 26086 Expires: 2027-03-27

Analyte:	Assigned Value	Units
Creatinine	146.7	mg/dL



Medica Corporation, 5 Oak Park Drive
Bedford, MA 01730-1413 USA



Emergo Europe, Prinsessegracht 20
2514 AP The Hague, The Netherlands